



**WARNING!**

**Risk of Entrapment**

- Pay attention to the distance between support and bed end.
- The safety distance (A) must be bigger than 390 mm.
- Pay attention to the distance between two support handles.
- The safety distance (B) when using two rails must be either 0-60 mm or 318 mm or bigger.
- Never use the support handles in combination with 3/4 side rails.

**Warning!**

**Risk Of Serious Injury**

- The support handles are not to be used as side rails; to keep the user from falling out of the bed.
- Only use the support handles as support when getting in and out of bed.

**User Instructions for Support Rails**



Rotate the metal plates out to the side. (Fig 2)  
Place the brackets onto the top of the frame following the diagram (fig. 1) and instructions outlined in the warning above.  
Rotate the metal plates back to close the brackets around the frame. (Fig. 3)  
Tighten the finger screws.

